


SanipullBulk Premium™ Quick Installation Guide for Nurse Call Pulls


Serrated bandage scissors and 1/4" *medium-duty* hand punch (available for purchase through your distributor as a kit) are required for standard ribbon and recommended for anti-ligature ribbon to minimize waste.

Attaching to Fixture

- To anchor the ribbon to the adaptor, punch a hole about 1/4" from the end of the ribbon or, if installing our anti-ligature ribbon, cut the ribbon about 1/4" adjacent to an existing hole. Fit the hole over the open adaptor's boss (Fig. 1). Close the lower door of the adaptor to secure ribbon.
- Cut the nurse call fixture's original cord or chain about 4" from the wall fixture. If a cord or string, tie a figure 8 knot at the end of it and cut off excess cord.

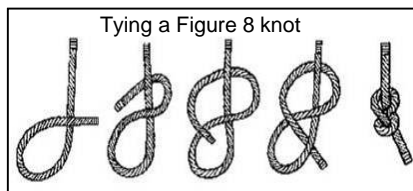

Optional grab handle can be positioned anywhere on the ribbon

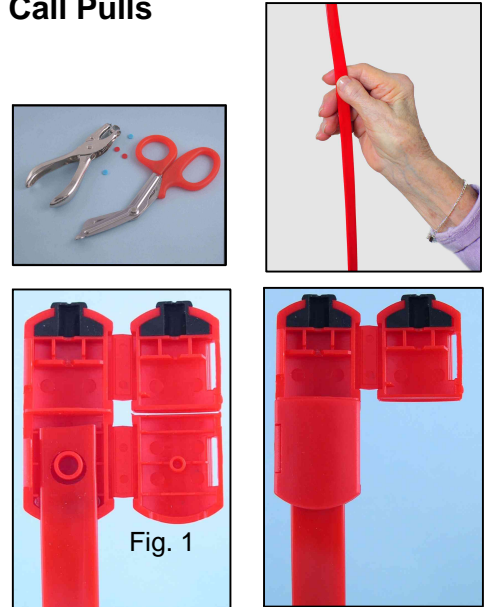

- Position the knot, terminal ball of ball or bead chain or melted ball end of Austco™ plastic lead into the clamshell adaptor (Fig. 2), taking care that it sits below the V-shaped notch. Close the upper clasp door.



Cutting the ribbon

Withdraw from the spool or coil sufficient Sanipull ribbon to reach about 4" from the floor. Cut the ribbon.




Attaching a handle




- Slide a cover (or two, if two handles used) onto the ribbon from the bottom end.
- Weave ribbon around the central post of the handle (Fig. 3), allowing at least 3/8" of ribbon to extend below the bottom post. (A second handle can also be installed mid-ribbon).
- Slide the cover over the posts to secure the handle.

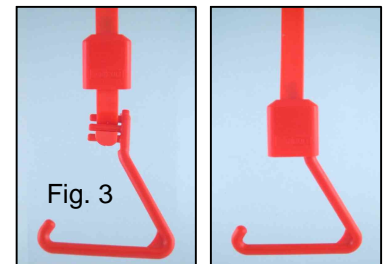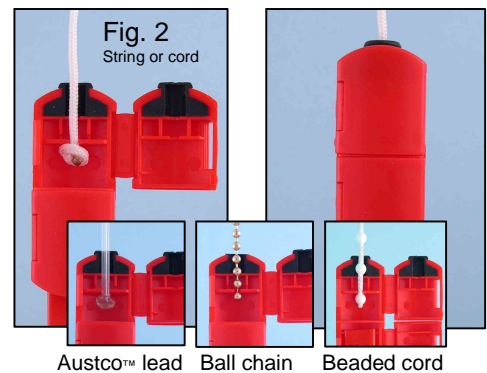

Attaching a SuperSanipull™ spring clip

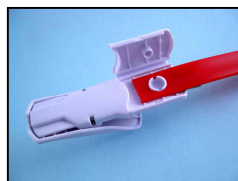

- Punch a hole about 1/4" from the end of the ribbon.
- Fit hole over the open clasp's boss. Close clasp door.
- Attach clip onto linen or clothing

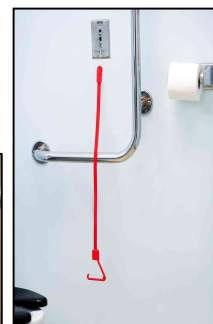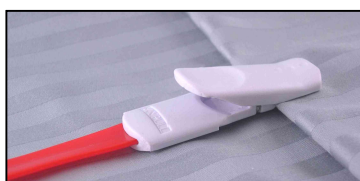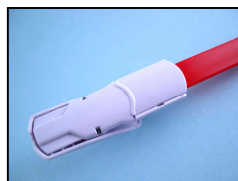

Sanitize Sanipulls after installation and clean regularly according to facility housekeeping policy.