


SanipullBulk Premium™ Quick Installation Guide for Over Bed Light Pulls

Assembly (~20 seconds):

1. Withdraw sufficient Sanipull ribbon to provide the optimal length between the fixture's pull chain to about 4" off the floor. Standard ribbon has tick marks every 12 inches for easier measuring.
2. At this length cut the ribbon with bandage scissors. If you are installing Sanipull's anti-ligature ribbon cut it ¼" adjacent to the closest hole.
3. A hole is required to anchor the clip to the ribbon. If necessary punch an anchor hole ¼" from the end of the ribbon using a medium duty hand punch (available from your distributor or directly from Wilshire Works).
4. Using the holes at the ends of the ribbon as anchor points fit the hole over the open clasp's boss and clamshell shut.



1000' spools of Sanipull ribbon

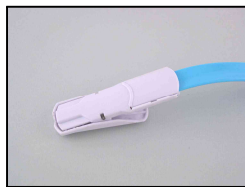


1/4" medium duty single-hole hand punch and bandage scissors make installation easy

5. Attach a SuperSanipull spring clip to each end of the ribbon pull.
6. The pull is ready to install onto the overbed light fixture.

Please Note: Consult the SuperSanipull Bulk Premium Price Matrix if you need additional ribbon (1000' spools, 360' spools or 120' coils) or SuperSanipull clips or grab handles



Ergonomic design is easy to see and grab

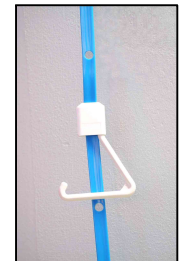

Optional grab handle can be positioned anywhere on the ribbon

Installation (~10 seconds):



1. Clip one of the pull's SuperSanipull spring clips onto the fixture's switch ball chain ensuring that one or two balls are engaged in the jaw's teeth. This attachment will provide a consistent release force of 12+/-1 lbs. If you would like to increase the release force to 15 +/- lbs we can provide a special adapter.

2. The sister clip on the bottom of the pull is attached around the bed rail in a convenient location by looping and clipping back onto the ribbon.

Note: This attachment method will provide optimal fixture protection from damage due to over force pulls. Anyone can reattach the spring clip if it becomes detached for ongoing switch protection. There is no need to call a maintenance person for reattachment.

**** Keep instructions in your Sanipull dispenser box.**

Optional Anti-ligature Ribbon Safety Enhancement: This modification provides weak zones every 6 inches along the length of the pull that allows the ribbon to tear apart when pulled with an over-force of 27 +/- 2 lbs. This option helps to minimize the potential for a pull to be used as a strangulation device. **Please note:** Wilshire Works Inc. will not be held liable for death or damages directly or indirectly involving any Sanipull product.

SuperSanipull clips are designed to mate only with Sanipull ribbon. Product warranty is void and switch protection will be compromised if clips are used with a non-approved pull medium.